



u-sonic

Electronic cleansing station for ear molds.

INSTRUCTION MANUAL

| | |
|----|-----------------------------|
| EN | INSTRUCTION MANUAL |
| DE | BEDIENUNGSANLEITUNG |
| FR | MANUEL D'UTILISATION |
| ES | MANUAL DE INSTRUCCIONES |
| PT | MANUAL DE INSTRUÇÕES |
| IT | MANUALE DI ISTRUZIONI |
| NL | GEBRUIKSAANWIJZING |
| TR | KULLANIM KILAVUZU |
| DK | BRUGERVEJLEDNING |
| FI | KÄYTTÖOPAS |
| SE | ANVÄNDARHANDBOK |
| NO | BRUKSANVISNING |
| HU | HASZNÁLATI ÚTMUTATÓ |
| PL | INSTRUKCJA OBSŁUGI |
| CZ | UŽIVATELSKÁ PŘÍRUČKA |
| SL | NÁVOD NA POUŽITIE |
| BG | РЪКОВОДСТВО ЗА ЕКСПЛОАТАЦИЯ |
| RU | РУКОВОДСТВО ПО ЭКСПЛУАТАЦИИ |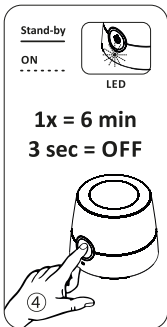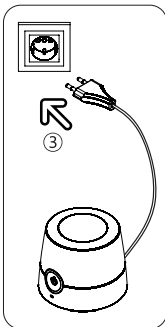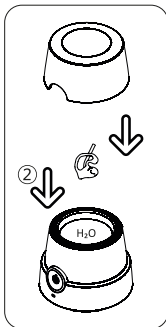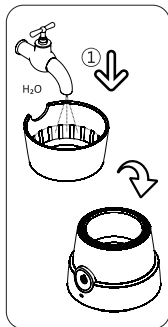

EN INSTRUCTION MANUAL

Perfect cleansing and care by power-ultrasonic

Dirt and ear wax can impair the function of ear molds, hearing protection, slim tubes and cause failures. The u-sonic offers optimal cleansing. Powerful ultrasonic at 46 kHz ensures already with clean water hygienic cleanliness.

Operation

1. Fill the cover of the u-sonic to the mark with water and pour it into the stainless steel tub
2. Place the to be cleaned parts into the liquid and close the cover
3. Plug the power supply into an AC power socket
 - ① LED permanent light
4. Briefly touch the sensor button and the ultrasonic cleansing program is starting for 6 min
 - ① LED blink light
5. The device automatically shutdown to stand-by after ultrasonic cleansing process
 - ① LED permanent light
6. Open the cover and remove the cleaned parts
7. Drain the water into a sink after each cleaning process

Maintenance and operation

- Switch off and unplug the appliance before any cleaning and maintenance.
- After ultrasonic cleansing, rinse the cleaned parts under fluent water. Tubes and holes must be well blown and dried.
- Simple cleansing can be done with pure water. For intensive cleansing a cleansing concentrate from audiologists or a splash of mild dish detergent is recommended as an additive.
- You can manually switch the device into stand-by during cleansing process by holding the sensor button for more than 3 seconds.
- We recommend wet wipes for cleaning your u-sonic.

Main application

- Ear molds, hearing protection, slim tubes and domes

Other application

- Dentures and braces
- Jewelry, rings, earrings, necklaces, bracelets that are cleanable by ultrasonic;
Ask your jeweler!
- Coins, medals, badges, razor shaving heads, pen nibs

Warnings

• CAUTION: Pull or unscrew hearing aids from the ear molds. Never put hearing aids into the liquid! • CAUTION: Use only with ultrasonic cleanable parts! • CAUTION: Don't use the u-sonic near sinks etc. - Risk of electric shock! • CAUTION: Never operate the u-sonic without water in the stainless steel tub! • In the event of a malfunction, unplug the power supply and take the drying station to your retailer. • Don't disassemble the device. • This appliance can be use by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazard involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision. • If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard. • Continuous running of the appliance should be not more than 12 minutes (Kb12min.) with the appliance cool down to room temperature before switching on again. • Follow Waste Electrical & Electronic Equipment (WEEE) instructions on page 38.

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EN - Do not dispose of electrical appliances as unsorted municipal waste; use separate collection facilities. • Contact your local government for information regarding the collection systems available. • If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being.

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